

Black Perching Stool With Padded Back
TRP-90273



Assembly Instructions and Guide

Thank you for choosing The Ramp Factory. Please read this guide carefully prior to use.

Intended Use

The perching stool is designed to be used as a mobility aid.

Unpacking

When unpacking the mobility aid please check you have all the required parts and fixings and contact The Ramp Factory if any items are missing. Please check the item for damage, if any damage is noted please contact the supplier immediately. Do not use a damaged item.

Product Specification

Width	560mm – 620mm
Depth	405mm – 420mm
Seat Width	368mm
Seat Depth	278mm
Seat Height	535mm – 680mm
Width Between Arms	510mm
Height of Armrests From Seat	175mm
Capacity	165kg (26 St)

The Ramp Factory
hello@therampfactory.co.uk
Tel: 01372 648587

Unit 5 Quantum Business Park, Beacon Hill Road, Fleet GU52 8EA

Assembly

To adjust the height, remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg.

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and that it fits neatly and snugly around the outside leg.

Note the perching stool is designed to slant to the front. Do not adjust any of the legs to make the seat level as this will cause excessive stress to the legs compromising user safety and possibly leading to permanent damage of the perching stool, including fracture of the tubular and other sections.

Cleaning

Prior to cleaning the chair please remove all pin clips and the four extension legs with the ferrules fitted. Clean these items separately to the main frame.

Clean the upholstery using a non-abrasive soft cloth with a mild detergent. Abrasive cleaners and/ or abrasive cleaning pads will seriously damage the upholstery beyond repair.

Using the Stool Safely

- When moving always lift the perching stool by placing your hands on the tubular metal frame beneath the seat
- Do not allow children or unauthorised persons to play with the equipment or use without proper supervision.
- Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury.
- Do not use the perching stool outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury.
- Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.
- If you are unsure if a Perching Stool is right for you we recommend consultant your doctor or Occupational Therapist
- Ensure there is adequate space for the stool so that it does not cause trip hazards
- Do not lean over the side of the stool or reach down towards the floor
- We do not recommend using a perching stool if you have poor balance or control of your upper body when sitting
- The stool should not be used by users who cannot put any weight on one or both legs

The Ramp Factory
hello@therampfactory.co.uk
Tel: 01372 648587

Unit 5 Quantum Business Park, Beacon Hill Road, Fleet GU52 8EA

Maintenance

Periodically check to ensure all components are present, correctly assembled and suitably tightened.

Warranty

The supplier provided a 1 year warranty against defects from the original purchase date. Please note the warranty will be void if the shower stool is used outside of the recommended guidance. No warranty is provided if the mobility aid is used for purposes other than as a mobility aid. If you wish to undertake a warranty claim please contact us and provide details of the fault and images via email at hello@therampfactory.co.uk. The Ramp Factory are not responsible for incidental or consequential damages.

Returns

The Ramp Factory offer a 60 day returns period if the product is not suitable for your needs. The item needs to be returned in the original packaging to The Ramp Factory. Please ensure to package the item correctly as we cannot be held liable for damage caused in transit. Please contact us if you wish to return the item for full returns details.

The Ramp Factory
hello@therampfactory.co.uk
Tel: 01372 648587

Unit 5 Quantum Business Park, Beacon Hill Road, Fleet GU52 8EA